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GEOTEK® NEPHROSTOMY BALLOON DILATOR**ENG****Instruction for Use**

Indications: intended for Dilation of path to Kidney. Made of Medical grade PA to provide little rigidity for stress applied for dilation. There are platinum makers at both ends of balloon.

Contraindications: Infections, obstruction, incontinence

Potential Complications and risks: Potential damages of urethra, perforation and bleeding.

Cautions: This Kit should be used by only by physicians experienced in the use of such device.

Ethylen oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30°C temperature, protect from sun light. Possible allergic reactions should be considered. **For single Patient Use Only.** Do not attempt to clean or resterilize this product. Disposal as a biohazard material after use

Procedure:

- The kidney is penetrated with a access needle.
- A guidewire is sent through the needle ,
- When the G.wire is placed, remove the needle, with firmly holding the wire.
- Placed the balloon Catheter in desired area and inflate.
- After dilatation, deflate and remove the ballon catheter

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

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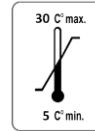
Single Use Storage limits Expiry Date

Ethylen oxide sterilized

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.ŞTİ